beurer

MG 520 To Go

D	Shiatsu-Massagekissen Gebrauchsanweisung.....	I	Cuscini massaggianti shiatsu Istruzioni per l'uso
	2		31
GB	Shiatsu massage cushion Instruction for Use	TR	Shiatsu Masaj Yastığı Kullanım Kılavuzu.....
	10		38
F	Coussin de massage Shiatsu Mode d'emploi.....	RUS	Массажная подушка шиatsu Инструкция по применению ...
	17		45
E	Cojín de masaje Shiatsu Instrucciones de uso	PL	Poduszka do masażu shiatsu Instrukcja obsługi
	24		53

Inhalt

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Germany

Contents

1. About Shiatsu massage	11	6. Initial use	15
2. Getting to know your device	11	7. Operation.....	15
3. Intended use.....	11	8. Maintenance and storage.....	16
4. Safety notes.....	12	9. What if there are problems?	16
5. Device description.....	14	10. Technical data	16

Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

Included in delivery

- Shiatsu massage cushion
- Mains part
- These instructions for use

WARNING

- The device is intended only for domestic/private use, not for commercial use.
- This device may be used by children over the age of eight and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

Signs and symbols

The following symbols appear in the instructions for use and on the type plate:

WARNING Warning instruction indicating a risk of injury or damage to health.

IMPORTANT Safety note indicating possible damage to the unit/accessory.

Note Note on important information.

The unit is double-protected and thus corresponds to protection class 2.

Only use indoors.

1. About Shiatsu massage

Shiatsu is a type of body massage that developed in Japan over 100 years ago, based on ideas taken from traditional Chinese medicine. The basic principle is that there is a system of energy channels (meridian system) within the human body.

The objective of a Shiatsu massage is to boost the physical, emotional and mental well-being of the treated person. To this end, energy blockages in the energy channels must be removed and the self-regulating powers of the body must be stimulated.

Shiatsu therapy achieves this through the application of pressure in flowing movements along the energy channels (meridians). Although Shiatsu literally translates as “finger pressure”, treatment is not just given with the fingers but also with the heels of the hand, elbows and knees. The therapist uses their whole body weight to create the necessary pressure.

2. Getting to know your device

The rotating massage heads in your Shiatsu massager imitate the pressure and kneading movements of a Shiatsu massage.

Using this electric Shiatsu massager, you can provide yourself or others with a soothing, effective massage with no help required. Massages can be relaxing or stimulating and work effectively to relieve strained muscles, aches and fatigue. The device offers a powerful, intensive massage for the neck, legs, arms and back.

The device is suitable only for **private use**, and not for medical or commercial purposes.

The Shiatsu massager also offers you the following benefits:

- Easy operation
- High-performance lithium-ion battery
- Heat and light function

3. Intended use

This device is intended to massage individual parts of the human body. It cannot replace medical treatment. Do not use the massager if one or several of the following warnings applies to you. If you are unsure whether the massager is suitable for you, please consult your doctor.

WARNING

Do not use the massager

- On animals
- If you suffer from a medical abnormality or injury in the area to be massaged (e.g. herniated disc, open wounds)
- Under any circumstances for massages in the area around the heart if you wear a pacemaker. If this is the case, consult your doctor regarding other parts of the body first
- On swollen, burnt, inflamed or damaged skin and body parts of any kind

- On cuts, capillaries, varicose veins, acne, couperose, herpes or other skin diseases
- During pregnancy
- On the face (eyes), on the larynx or any other particularly sensitive body parts
- For foot massages
- While sleeping
- After the consumption of medicines or alcohol (reduced cognition!)
- In vehicles

The device has a hot surface. Persons sensitive to heat must be careful when using the device.

You should consult your doctor before using the massager, especially

- if you suffer from a severe illness or have recently had surgery on your upper body,
- if you have a pacemaker, implant or other aids,
- if you suffer from thrombosis,
- if you suffer from diabetes,
- if you are suffering pain, the cause of which is unknown.

The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

4. Safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the device to third parties.

WARNING

Keep packaging material away from children to avoid danger of suffocation.

Electric shock

WARNING

As with every electrical device, use this massager carefully and cautiously, in order to avoid hazards due to electric shock.

Therefore, observe the following instructions for use:

- Use the device only with the mains part provided and at the mains voltage specified on the mains part
- Never use the device if it or its accessories show visible signs of damage.
- Do not use the device during a thunderstorm.

In the event of defects or operational faults, switch the device off immediately and disconnect it from the mains socket. Do not pull on the mains cable or on the device to disconnect the mains part from the socket. Never hold or carry the device by the mains cable. Keep the cables away from hot surfaces. Do not crush, bend or twist the cables. Do not insert needles or any sharp objects.

Ensure that the massager, control panel, mains part and cable do not come into contact with water, steam or other liquids. Only use the device in dry rooms (e.g. never in the bath, sauna).

Never reach for a device that has been submerged in water. Disconnect it from the mains plug immediately.

Do not use the device if it or its accessories are visibly damaged.

Do not drop the device or subject it to any impacts.

Notes on handling rechargeable batteries

- If your skin or eyes come into contact with fluid from the battery cell, flush out the affected areas with plenty of water and seek medical assistance.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the rechargeable batteries.

- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery prior to initial use (see Chapter 6).
- In order to achieve as long a battery service life as possible, fully charge the battery at least twice per year.

Repairs

WARNING

- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.
- The zip fastening of the Shiatsu massager must not be opened. It is only used for technical production reasons.

Risk of fire

WARNING

There is a risk of fire if the device is used improperly or these instructions for use are ignored.

Therefore

- never use the massager underneath a cover, such as a blanket, pillow, etc.
- never use the device near petrol or other highly flammable substances.

Handling

IMPORTANT

The device must be switched off and disconnected from the mains after every use and before cleaning.

- Do not insert any objects into the device apertures or the rotating parts. Ensure that the moving parts can move freely at all times.
- Do not place your full weight on the moving parts of the device or place any objects on the device.
- Do not expose the device to high temperatures.

Disposal

IMPORTANT

- The empty, completely flat rechargeable batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the rechargeable batteries.
- Remove the battery from the device by opening the zip fastener and removing the cover as far as possible. Unscrew all of the screws on the housing, and remove the housing cover. The battery is located under the massage heads.
- The codes below are printed on rechargeable batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).


If you have any questions, please contact the local authorities responsible for waste disposal.

The device battery must be disposed of separately. The battery must not be disposed of with domestic waste.


5. Device description

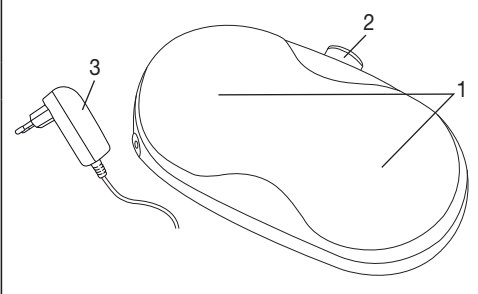
Shiatsu massage cushion (front)

	1. 4 massage heads (2 heated and illuminated massage heads), rotating in pairs
	2. Control panel
	3. Mains part

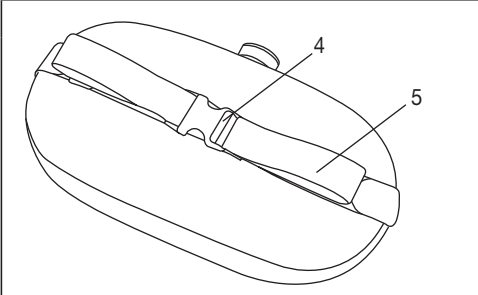Shiatsu massage cushion (rear)

	4. Clip buckles for attachment
	5. Handle

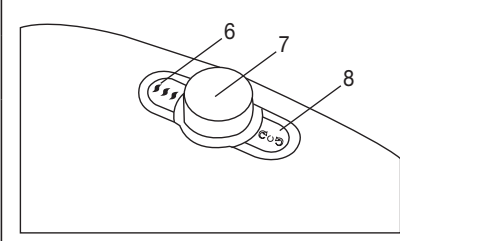Control panel

	6. Optional light and heat function
	7. Dial for adjusting the widths of the massage heads
	8.  On/Off button   Clockwise or anti-clockwise rotation of the massage heads

6. Initial use

- Remove the packaging.
- Check the device, mains part and cable for damage.
- Insert the connector plug into the socket on the massager to charge the device.
- Charge the device for at least 3 hours before initial operation. The on/off LED flashes blue on the control panel during charging.

As soon as the device is fully charged, the LED light lights up blue.

When the battery is flat, the LED light flashes red.



The device can also be used during charging. However, this makes charging slower.

- Ensure that the cable does not pose a trip hazard.

7. Operation

At first, just press the body area that you want to massage carefully against the massager. Then check carefully whether the massage position is pleasant for you and then lower your weight gradually towards the massager. Set the distance between the massage heads using the dial [7]. To do this push the dial downwards to disengage the lock and turn the dial left or right at the same time.


The massage should feel pleasant and relaxing at all times. If the massage is painful or uncomfortable, stop the massage or change your position or the contact pressure.

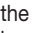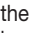- Do not force or jam any part of your body, in particular your feet, hands and fingers, between the rotating massage heads or their holders in the device.
- Switch on the massager using the  button. The massage heads start to rotate. Press the  button again to change the massage direction. Pressing the button again turns the device off. Press the  button to activate the light and heat function. Press the  button again to turn off the light and heat function.

Note


- The cushion can be attached to a chair using clip buckles.
- You can also use the cushion using the handles.
- Use the massager for max. 15 minutes. Longer massages can lead to over-stimulation of the muscles, creating tension instead of aiding relaxation.

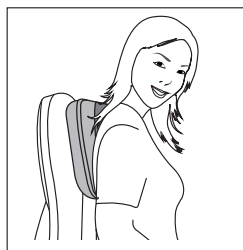
Back massage: upper back area

Back massage: lower back area (lumbar)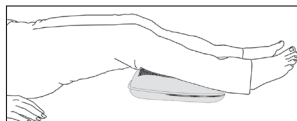

Leg massage

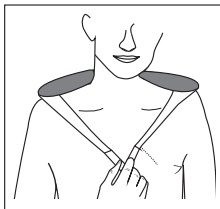Back massage: upper back area

8. Maintenance and storage

Cleaning

WARNING

- Always unplug the mains part from the socket and disconnect it from the device before carrying out any cleaning.
- Clean the device using only the methods specified. Under no circumstances may liquid enter the device or the accessory.
- Small marks can be removed using a cloth or damp sponge, and some liquid delicate laundry detergent where necessary. Do not use any solvent-based cleaning products.
- Do not use the device again until it is completely dry.

Storage

If you do not plan to use the device for a long period of time, we recommend that you store it in the original packing in a dry environment, ensuring it is not weighed down by additional items.

9. What if there are problems?

Problem	Cause	Solution
Massage heads rotating too slowly.	The load on the massage heads is too great.	Free up massage heads.
Massage heads do not move.	Device is not connected to the mains or battery power is too low.	Charge the device.
The widths cannot be adjusted.	Rotary knob for adjusting the width is not pushed down.	Push and hold the dial downwards and adjust the distance between the massage heads by turning it left or right.

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Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany